

## Agma Cast Orthopedic casting tape

### Advantages

- The tape combines knitted fiberglass (or polyester fabric) with water-cured resin.
- Light Weight - It is 5 times lighter and stronger than plaster cast.
- Air Permeability - Air can be ventilated well through the porous surface, so it helps patient not to feel itching on applied region.
- Easy To Mold - It is easy to mold on the region due to its softness and elasticity.
- Excellent Conformability - AgmaCast features multi-directional extensibility to conform well to contours, improving cast fit and appearance.



### Certificates

ISO 13485

### Application

1. Apply stockinette and undercast patting.
2. Wear gloves to prevent resin adhering to the skin.
3. Dip the product in water for 3 or 5 seconds and squeeze water from the product.
4. Overlap the product on the site with an half or two third of its width as the spiral shape.
5. Overlap 2-3 or 3-5 layers on the site.
6. Molding time is required about 3-5 minutes and weight bearing 20 minutes.

### Product Line-up

Model	Size		Packing
MAC-2000 (White/Green)	2inch x 4yds	5cm x 360cm	10 rolls / Inbox 10 inboxes / Carton
MAC-3000 (White/Green)	3inch x 4yds	7.5cm x 360cm	

Model	Size		Packing
MAC-4000 (White/Green)	4inch x 4yds	10cm x 360cm	10 rolls / Inbox 10 inboxes / Carton
MAC-5000 (White/Green)	5inch x 4yds	12.5cm x 360cm	

+ Shelf life (unopened) : 2 years

## Agma Splint Orthopedic splint

### Advantages

- Bacteriostatic and deodorant effects
- Variously classified products - One-Step & Roll Type : 11 kinds
- Strong Weight Bearing - It can prevent from twisting in long-leg treatment.
- Comfortable Feeling - It will give you comfortable feeling by adopting high-quality non-woven fabric
- Good appearance, Easy application
- Light weight, Economical polyurethane resin



### Certificates

ISO 13485

### Application

1. Choose the proper size product to the patient
2. Open the product, In case of the roll product, cut the roll product with scissors properly.
3. Dip splint in the water for 3 or 5 seconds.
4. Squeeze out wet splint with a towel in order to eliminate remaining water.
5. Apply the splint to the fractured limb and wrap with elastic bandage.

### Product Line-up

	Model	Size		Packing
One-Step Splint	MPO-3120	3inch x 12inch	7.5cm x 30cm	10 pcs / Inbox 4 inboxes / Carton
	MPO-3350	3inch x 35inch	7.5cm x 89cm	
	MPO-4150	4inch x 15inch	10cm x 38cm	
	MPO-4300	4inch x 30inch	10cm x 76cm	
	MPO-5300	5inch x 30inch	12.5cm x 76cm	
	MPO-5450	5inch x 45inch	12.5cm x 115cm	5 pcs / Inbox 3 inboxes / Carton

	Model	Size	
Roll Splint	MPR-2080	2inch x 5yds	5cm x 460cm
	MPR-3080	3inch x 5yds	7.5cm x 460cm
	MPR-4080	4inch x 5yds	10cm x 460cm
	MPR-5110	5inch x 5yds	12.5cm x 460cm
	MPR-6080	6inch x 5yds	15cm x 460cm

+ Shelf life (unopened) : 2 years